

## Care and Maintenance of the OS-Care LCS

If the LCS begins to lose its tack or when the fabric becomes soiled, wash it with a mild soap and water. (We recommend using Dawn as there are no ingredients to compromise the tack.)

Use a very soft-bristled brush to gently clean soiled areas on either side. Rinse and then dry with a hair dryer or let air dry. Tackiness will be restored.

If the edges of the LCS begin to fray, simply trim the frayed material with a sharp scissors.


## To store the OS-Care LCS Device

When not in use, there are several ways to store your device. You can place it on ...

1. ... a clean plastic sheet or plastic bag;
2. ... waxed paper; and/or
3. ... parchment paper.

Place the fabric side down on a hard flat surface, then lay your clean storage medium over it, then smooth it down completely.

You could even adhere the silicone side to a bathroom mirror. This will keep the silicone clean and preserve the tack until you are ready to use it again.

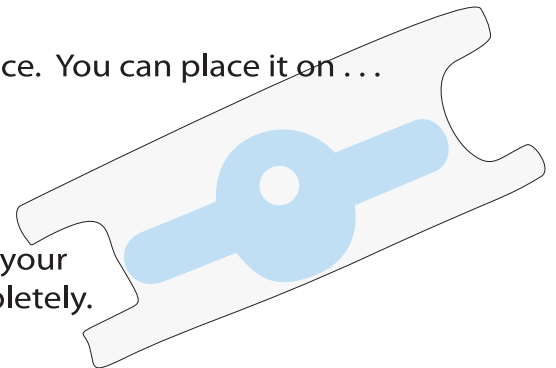

ULTRA



WOUND CARE

For questions or comments, please contact us

205 Powell Place, Brentwood, TN 37027 • 866-623-7476 • [info@DermaHug.net](mailto:info@DermaHug.net)